

E-connect S 手柄维修手册

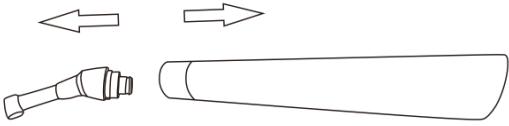
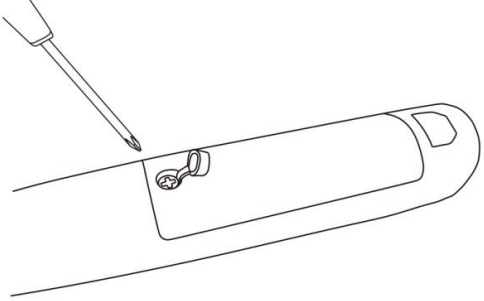
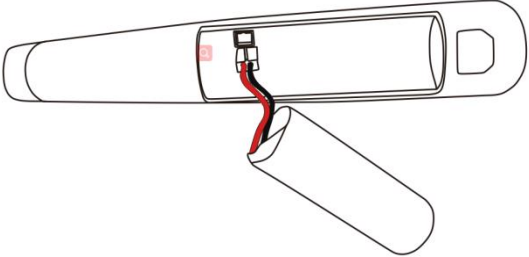

E-connect S Motor Handpiece Maintenance Manual

注意
Warning

此产品需要厂家授权的人员维修操作，否则保修失效
This product must only be repaired by manufacturer approved partner, Otherwise the warranty will be invalid.

拆卸

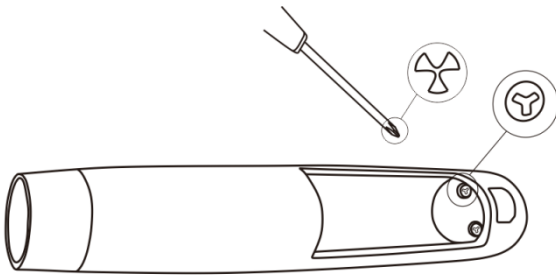
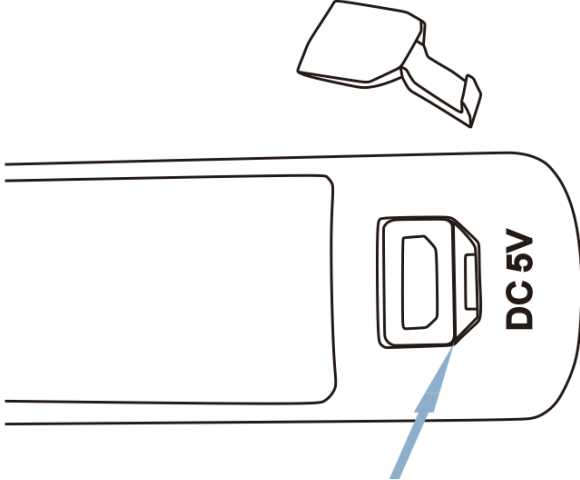
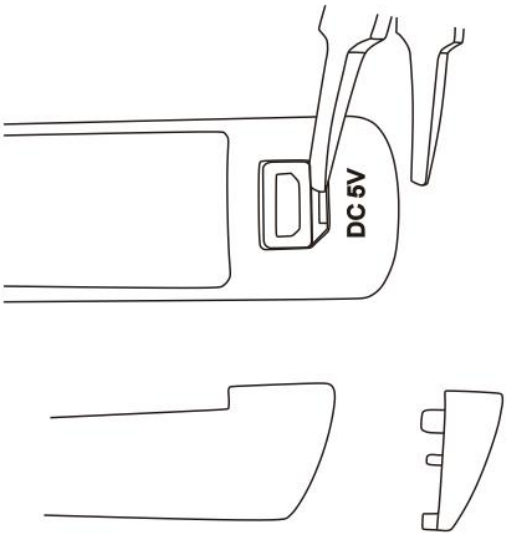
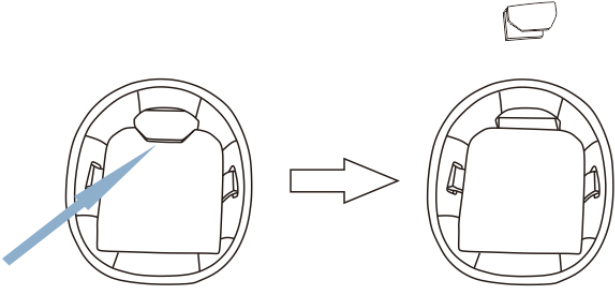
Disassembly

| 步骤 Step | 示意图 Diagrammatic Sketch | 说明 Description |
|------------|---|--|
| 1. |  | 拔出机头 Pull off the Contra Angle |
| 2 |  | 掀起电池盖上面橡胶盖，拧下螺丝，取下电池盖 Remove the rubber pad on the battery cover, screw off, and remove the battery cover |
| 3 |  | 取出电池，拔下电池插头。 Take out the battery. Unplug the wire. • |
| 4 |  | 撕下电池壳后端的白色泡沫垫 Remove the white foam. |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

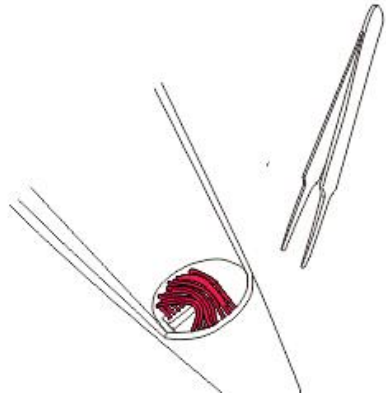
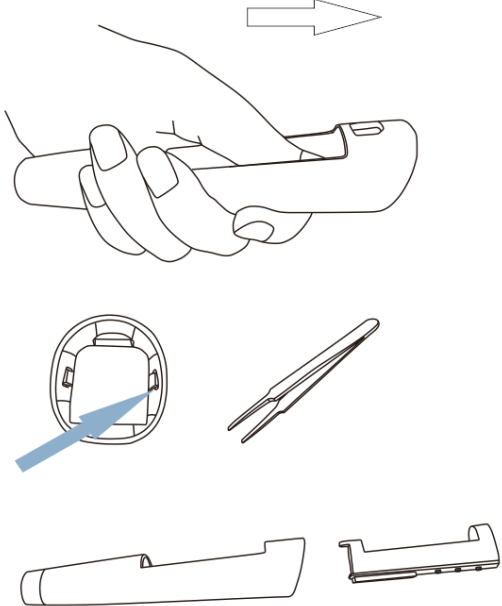
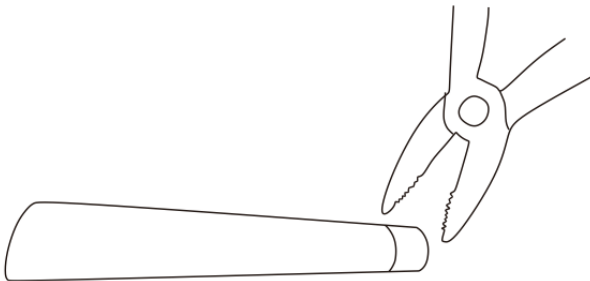
本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|---|---|---|
| 5 |  | <p>用图示三角头螺丝刀拧下电池壳后端两颗螺钉</p> <p>Unscrew the two screws at the bottom of the battery case with a triangular head screwdriver</p> |
| 6 |  | <p>取下 DC 端口的橡胶套</p> <p>Remove the rubber sleeve of DC port,</p> |
| 7 |  | <p>用镊子别图示后盖缝隙，取下后盖</p> <p>Insert tweezers into the gap between the back cover and charging cover of the drawing. Then remove the bottom cover</p> |
| 8 |  | <p>从图示位置顶出充电接口盖。</p> <p>Push out the charging interface cover from the position shown</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

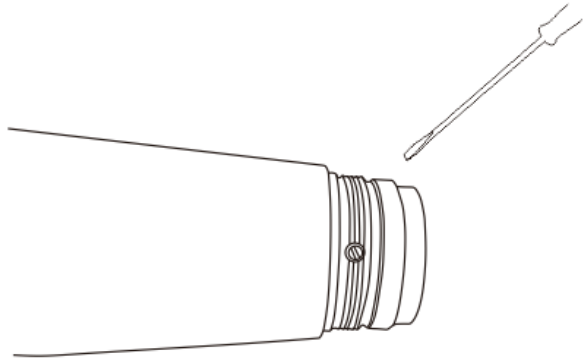
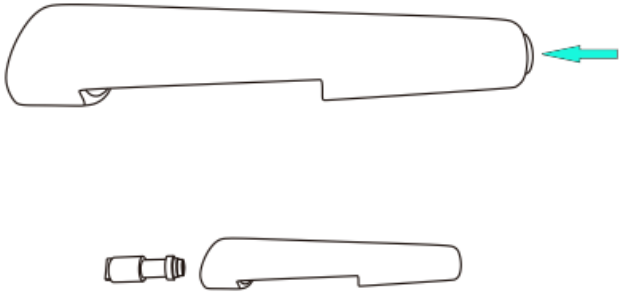
本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|----|---|--|
| 9 |  | <p>用镊子拔下连接马达的排线</p> <p>Use tweezers to remove the wires from the motor</p> |
| 10 |  | <p>用大拇指轻轻顶住电池壳，然后用镊子或者一字起撬起两边卡扣。推出电池壳及电路板，注意推出电路板时不要损坏按键</p> <p>Gently push the battery case with your thumb, then use tweezers or a Straight screwdriver to pry up two side clasps . Release the battery case and circuit board. Be careful not to damage the key when pushing out the circuit board</p> |
| 11 |  | <p>用老虎钳拧下头部螺母。注意：用老虎钳拆下螺母后表面可能有划痕，因此拧时需要垫上毛巾或纸巾，如划痕太深导致影响外观，需要更换</p> <p>Unscrew the head nut with a pliers。Note: there may be scratches on the surface after removing the nut with the pliers, so you need to pad the towel or paper when you screw the nut, If the scratch is too deep and affect the appearance, it needs to be replaced</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|----|---|--|
| 12 |  | <p>拆下马达接头处的三个锁紧螺钉。拆下螺套。 Remove the three locking screws from the motor joint. Remove the screws sleeve</p> |
| 13 |  | <p>从头部推出马达 Push the motor out from the front of the product</p> |

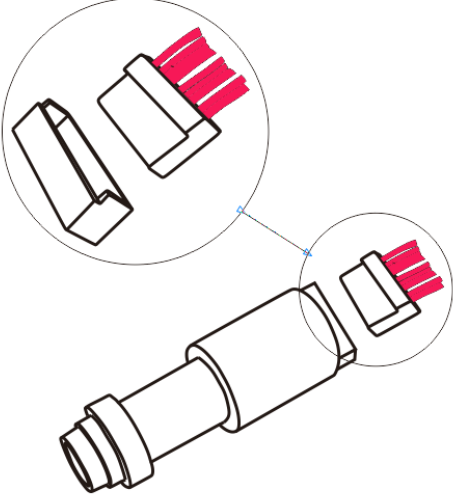
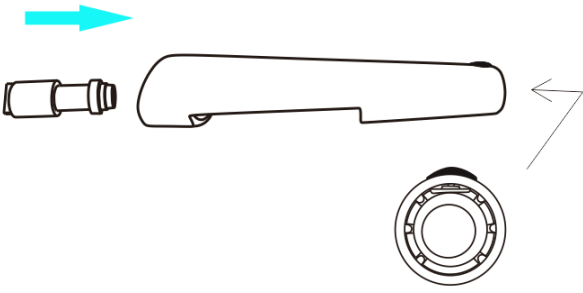
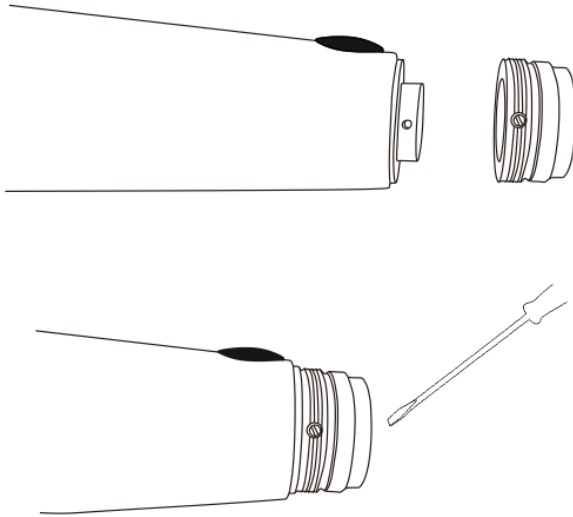
This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

安 装

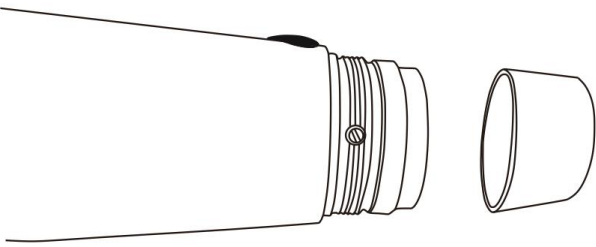
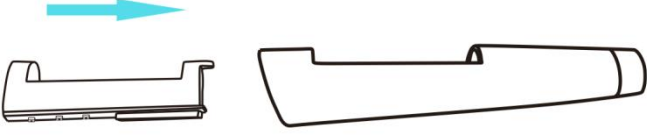
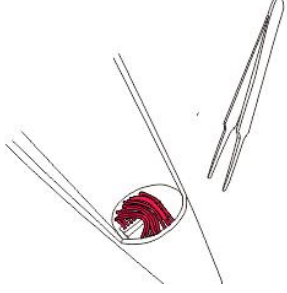
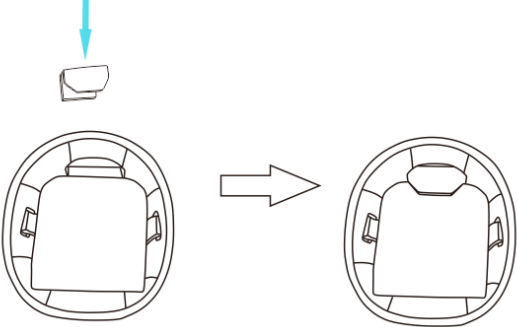
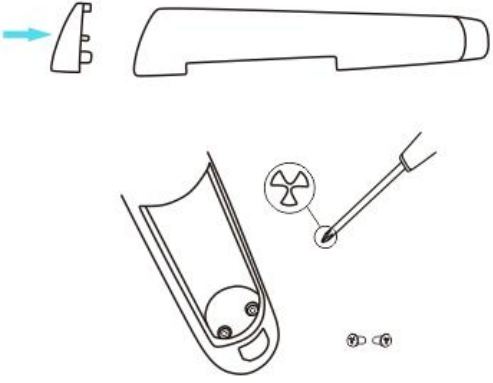
Installation

| 步骤 Step | 示意图 Diagrammatic Sketch | 说明 Note |
|------------|---|--|
| 1 |  | <p>安装马达排线，注意马达排线凸台对应马达插座缺口。</p> <p>Install the motor line. Please note that the protruding of the motor line needs to be aligned with the motor socket gap.</p> |
| 2 |  | <p>装马达。从尾部向前推出马达，注意马达上面凹槽对应外壳凸台，并且按键需要对应到按键套位置。</p> <p>Install the motor. Push the motor in from bottom. Please note that the grooves on the motor correspond to the convex of cover, and the key should correspond to the position of the button cover.</p> |
| 3 |  | <p>装螺套。螺套上的螺钉方向对准马达安装位置的小孔方向推入。锁紧紧定螺钉。</p> <p>Install screw sleeve. The screw in the screw sleeve is pushed into the hole on the motor. Screw up the screws</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

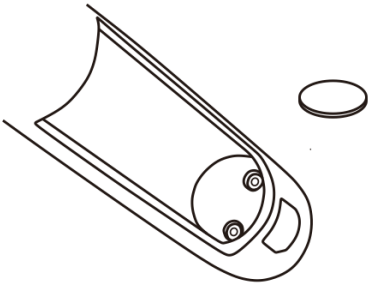
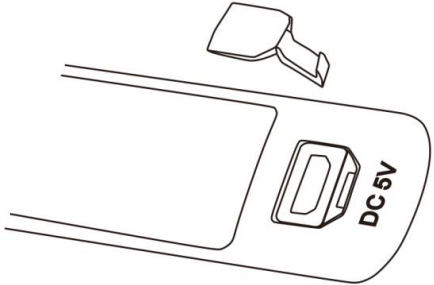
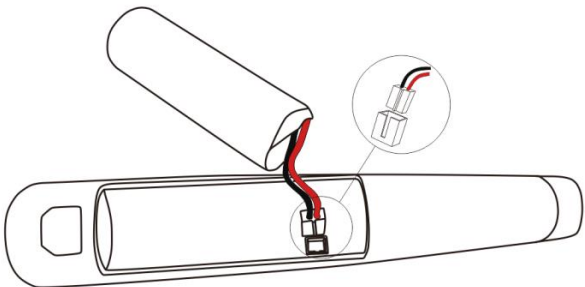
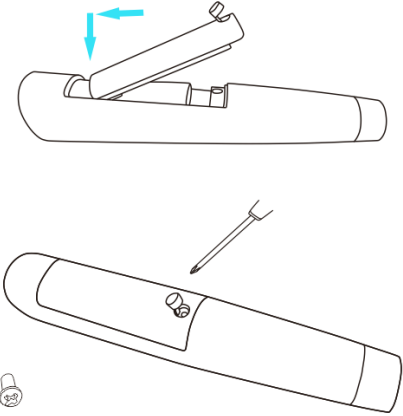
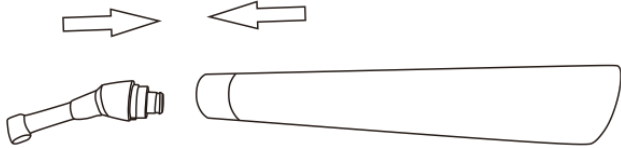
本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|---|---|--|
| 4 |  | <p>拧紧螺母 Tighten the nut</p> |
| 5 |  | <p>电池壳及电路板整体从后端推入，注意显示屏对准显示屏位置，推电池壳至后端卡扣卡到位置后停止。 Push the battery case and circuit board from the bottom of shell. Please note that the display screen is correspond to the display sticker of the shell. Push the battery case until the rear end of the battery case is stuck in the groove of the shell</p> |
| 6 |  | <p>安装马达排线。 Install the motor line to the circuit board</p> |
| 7 |  | <p>安装充电口盖 Install the charging cover</p> |
| 8 |  | <p>安装后盖并锁上螺钉 Install the bottom cover and lock the screws</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|----|---|--|
| 9 |  | <p>粘上白色泡沫垫 Stick the white foam pad</p> |
| 10 |  | <p>安装充电口橡胶套。 Install charging port rubber cover.</p> |
| 11 |  | <p>将电池插头插入电路板插座，注意电池插头上的凸台需要对准插座凹槽。 Insert the battery plug into the PCB socket. Notice that the socket on the battery plug needs to be aligned with the socket slot</p> |
| 12 |  | <p>安装电池盖。电池盖由前往后滑到位后向下压下。 锁上螺钉。 Install the battery cover. The battery cover slides into place and then presses down. Lock the screw.</p> |
| 13 |  | <p>装上弯机头。 Install the Contra Angle</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

E-connect S 底座维修手册

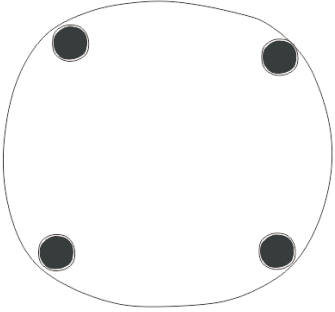
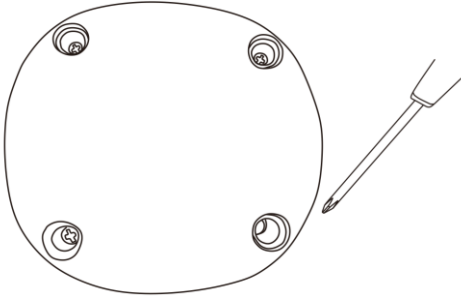
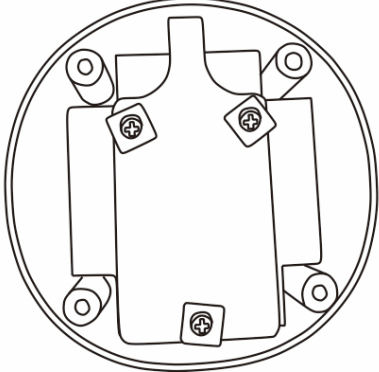
E-connect S Charge Basement Maintenance Manual

注意
Warning

此产品需要厂家授权的人员维修操作，否则保修失效
This product must only be repaired by manufacturer approved partner, Otherwise the warranty will be invalid.

拆 卸

Disassembly

| 步骤 Step | 示意图 Sketches | 说明 Note |
|------------|---|---|
| 1. |  | 取下底部四处黑色橡胶垫 Remove the black rubber pads around the bottom |
| 2 |  | 用十字螺丝刀拆下底部橡胶垫下四处螺丝，拆掉上盖 Remove four screws from the bottom of the base with a cross screwdriver, then remove the top cover |
| 3 |  | 拆掉电路板上三处螺钉。拆下方形垫片、电路板、圆环形橡胶垫片、配重铁块。 Remove three screws from the circuit board. Remove square spacer, circuit board, circular rubber gasket, Counterweight iron |

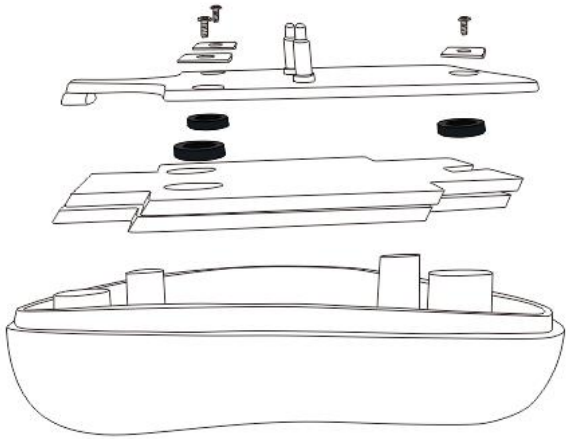
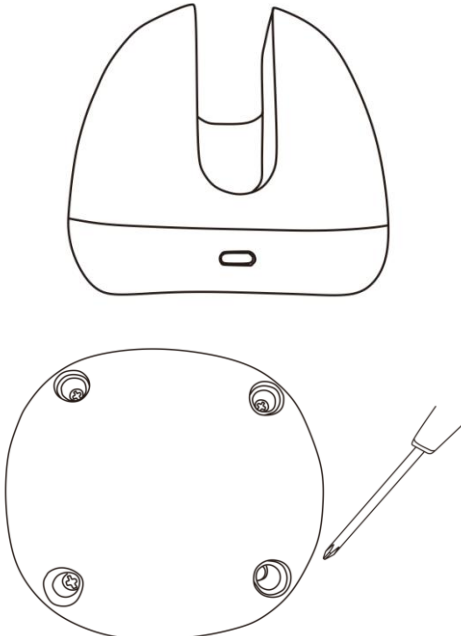
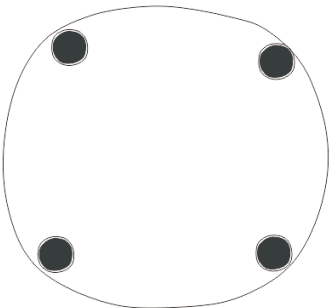
This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

安 装





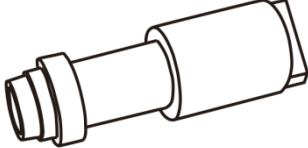



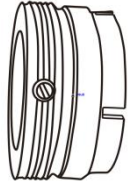
Install

| 步骤 Step | 示意图 Diagrammatic Sketch | 说明 Note |
|------------|---|--|
| 1 |  | <p>依次装上配重铁块，圆环形橡胶垫，电路板。方形垫片，打上螺钉。</p> <p>Install iron sheet, circular rubber pad, circuit board, square washer, screw</p> |
| 2 |  | <p>盖上上盖，注意上盖缺口方向对应底壳充电口方向。打上底部四处螺钉。</p> <p>Cover the upper cover, note that the upper cover notch direction corresponds to the bottom shell charging port direction. Screw the base around</p> |
| 3 |  | <p>重新粘上底部橡胶垫。</p> <p>Attach the rubber pad at the bottom of the base</p> |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.


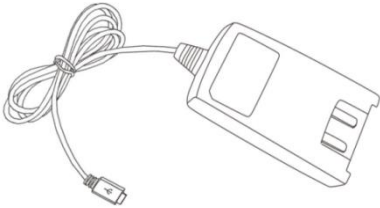
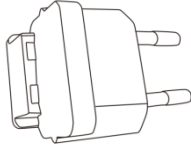

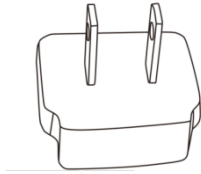

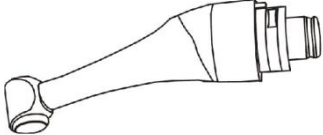
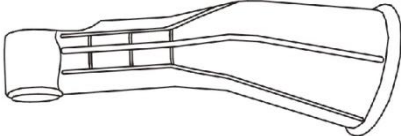
本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| 编号 Part number | 名称 Name | 图示 Diagrammatic Sketch |
|-------------------|---|---|
| 6014001 | 电池 18500 Battery |  |
| 6051033 | 带电路电池壳 Battery case with PCB and screen |  |
| 6051032 | 二代无电路手柄 Plastic shell |  |
| 6003103 | 一体电池盖 Battery cover |  |
| 6051031 | 二代电机组件 Electronic motor |  |
| 6021004 | 正 Y 槽盘头平尾自攻螺钉 ST2.2x6 Screw |  |
| 6021002 | 十字槽圆头平尾自攻螺钉 ST2X4 Screw |  |
| 6051034 | 手柄充电尾盖 Bottom cover |  |
| 6051041 | 电机夹紧件组件 Motor spare parts |  |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.



本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。


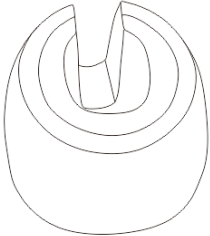
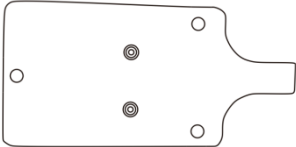


| | | |
|---------|---|---|
| 6005004 | 一体机 USB 盖 USB Cover |  |
| 6016007 | 开关电源无标准 Adapter (No standard) |  |
| 6016008 | 欧标头 Charging converter (European standard) |  |
| 6016009 | 英标头 Charging converter (UK standard) |  |
| 6016013 | 日标头 Charging converter (Japanese standard) |  |
| 6016010 | 澳标头 Charging converter (Australian standard) |  |
| 6036006 | E2 弯机头 Contra Angle |  |
| 6004027 | E2 机头套开口 Insulating Sleeve |  |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。

| | | |
|---------|-----------------------------|---|
| 6031009 | 一次性机头套 Disposable Sleeve |  |
| 6001009 | 外壳套 Shell metal sleeve |  |

| | | |
|---------|----------------------------------|---|
| 6003106 | 充电底座下壳 Charge basement bottom |  |
| 6003105 | 充电底座上壳 Charge basement upper |  |
| 6051040 | 二代底座带顶针电路 Charge basement PCB |  |
| 6021006 | 十字槽圆头平尾自攻螺钉 ST3X8 Screw |  |
| 6033025 | 一体机脚垫 Rubber pad |  |

This Maintenance Manual is CONFIDENTIAL AND PRIVATE INFORMATION of Changzhou Sifary Medical technology Co., Ltd.

Any unauthorized use, dissemination, forwarding, printing or copying of such information is strictly prohibited.

本维修手册为本公司保密和专有信息。未经本公司授权，严禁任何针对本手册的使用、传播、转发、打印、复制行为。