

Service Manual of CuringPen

1. Purpose

The maintenance manual is used to guide the dealer to repair the malfunctioning machine, reducing the time cost and labor cost of returning to the factory for repair. The manual provides the following information:

Failure analysis and maintenance video: List the failure phenomena of the machine, and guide the dealer to replace the machine parts through the maintenance video;

Replaceable repair parts: provide dealers with replaceable repair parts.

2. Scope of application

This specification applies to the company's corresponding products.

3. Content

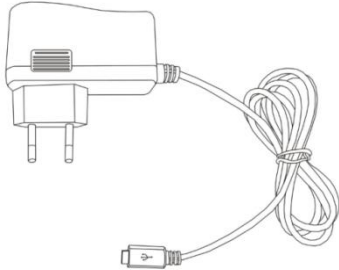
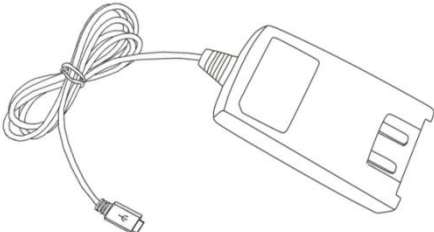
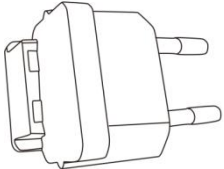
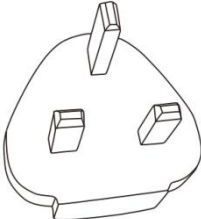
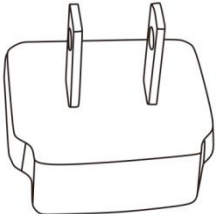

1) Failure analysis and maintenance video.




1.	Can't charge or can't turn on	Replace PCB assembly, check video "6580001-replace PCB assembly" for reference.	/
		Replace battery assembly, check video "6580001-replace battery assembly" for reference.	
		Replace back cover assembly, check video "6580001-replace back cover assembly" for reference.	
2.	<i>Show "Error"</i>	Replace Light Source Head.	
		Clean the PCB inside the Light Source Head(bottom part of the Light Source Head), check video"6580001-Clean the PCB inside the Light Source Head" for reference.	




3.	Still show "Error" after replacing Light Source Head	Replace PCB assembly, check video "6580001-replace PCB assembly" for reference.	
4.	Screen blurred	Replace PCB assembly, check video "6580001-replace PCB assembly" for reference.	

2) components for maintenance

Order code	Description	Pic for reference
6551009	Battery assembly	
6551002	Main PCB assembly	
6501006	back cover assembly	
6551007	Handpiece cover+printed board	

6516001	Adapter(EU plug)	
6516003	Adapter (without plug)	
6016008	EU plug	
6016009	British plug	
6016013	Japanese plug	
6016010	Australian plug	

<p>6551001</p>	<p>Light Source Head</p>	
<p>6551004</p>	<p>Charging base</p>	
<p>6551003</p>	<p>Handpiece assembly</p>	
<p>6542002</p>	<p>Disposable Sleeve</p>	
<p>6551005</p>	<p>Protective Light Shield</p>	

6503011	Depth test board	
6541001	Protective light glasses (polarized glasses)	
6503005	Gasket of PCB	
6043002	Sealant	